**IMPORTANT MEDICAL INFORMATION: RETAIN FOR FUTURE REFERENCE.
DENTAL IMPLANT SURGICAL KIT INSTRUCTIONS FOR USE**

Note: This Surgical Kit is supplied non-sterile and must be cleaned and sterilized prior to first use and each use thereafter. Remove and discard the plastic shipping material before initial sterilization.

CLEANING & STERILIZATION

1. Remove any visible debris from the instruments and surgical kit using a soft bristle brush and a broad spectrum cleaning or disinfecting agent such as Hu-Friedy's Enzymax® or equivalent (call 773.975.3975 to order). Rinse thoroughly.
2. Place the instruments in a beaker of the same solution and sonicate for 10 minutes. Rinse thoroughly.
3. Rinse instruments with isopropyl alcohol to remove any soap residue and minerals (important to help prevent corrosion).
4. Blot instruments with a lint-free towel and allow them to air dry completely.
5. Disassemble the surgical kit and wash the empty tray using the same solution. Rinse the tray with water and dry thoroughly.
6. Return instruments to the appropriate location in the surgical tray.
7. Wrapping the kit in a standard approved sterilization wrap, use one of three qualified steam sterilization cycles:
 - a. Prevacuum Steam: 132°C (270°F) for five minutes minimum.
 - b. Gravity Steam: 132°C (270°F) for thirty minutes minimum.
 - c. Gravity Steam: 121°C (250°F) for sixty minutes minimum.Dry for 20 to 50 minutes.

- Caution: Failure to completely dry surgical instruments during autoclaving can leave moisture and cause discoloration and oxidation.
- Caution: The use of hydrogen peroxide or other oxidizing agents will damage the surface of the instruments.
- Caution: Periodic testing, cleaning, and calibration of autoclave equipment is recommended to ensure the unit remains in proper working order.
- Caution: Drills and taps should be replaced when wear is noticed, such as a decrease in cutting efficiency or when signs of discoloration appear. Cutting instruments should be replaced after approximately 12 to 20 osteotomy cycles, depending on bone density; drills that cannot advance 2mm per second in the osteotomy have become dull and should be replaced.

FEDERAL LAW (US) RESTRICTS THIS DEVICE TO THE SALE, DISTRIBUTION, AND USE BY OR ON THE ORDER OF A DOCTOR OR DENTIST.

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